

Simply relevant for the most common fetal trisomies.



## Ordering the VisibiliT™ test

# A simple, quick-start guide

### WHO SHOULD GET THE TEST?

Patients who would benefit from the VisibiliT laboratory-developed test include those who:

- Have a single gestation pregnancy and are considered to be at average risk for fetal aneuploidy
- Want to know common genetic information
- Desire high accuracy for trisomy 21 (associated with Down syndrome), trisomy 18 (associated with Edwards syndrome) and fetal gender

### ORDERING

- Complete the test requisition form (TRF).
- In the OTHER section on the TRF:
  1. Handwrite in that area:  
VisibiliT Test NCCP

#### OTHER

*VisibiliT Test NCCP*

2. Or, mark this box, if it is preprinted:

#### OTHER

**VisibiliT™ test** (For singleton pregnancies only)  
Non-covered cash pay option. Insurance will not be billed.


### PATIENT PAYMENT

**The VisibiliT test is currently offered as a non-covered cash pay option and insurance will not be billed.**

Once the sample arrives at Sequenom Laboratories, a Billing Associate will contact the patient to request credit card payment for the VisibiliT test.

Sequenom®, MaterniT21® PLUS and VisibiliT™ are trademarks of Sequenom, Inc. and used with permission by Sequenom Center for Molecular Medicine, LLC, dba Sequenom Laboratories. ©2014 Sequenom Laboratories. All rights reserved.

### SAMPLE COLLECTION AND SHIPPING

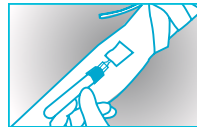

- Use either the Sequenom Laboratories all-in-one prenatal transport pack or the MaterniT21® PLUS transport pack.



- Fill the included 2 x 10 mL mottled black/tan top blood collection tubes and follow the provided handling and packing instructions.



- Pack and label patient sample according to provided instructions, along with a completed TRF, and return to Sequenom Laboratories for testing.

### LAB RESULTS

Results are provided as a personalized risk score and will be reported in approximately 5 days from receipt of sample in the lab. A lab report is provided for all results and, additionally, we will call your office to discuss all high-risk results.



### SUPPORT

For any questions regarding the VisibiliT test, please contact your local Business Development Manager or one of our friendly Customer Service team members at 877.821.7266 between 5 am and 5 pm Pacific time. You can also request assistance at any time by email at [clientservices@sequenom.com](mailto:clientservices@sequenom.com).

Toll Free (within the US) at

# 877.821.7266